

veterinary technology for life

Eickemeyer®

www.eickemeyer.co.uk

Telephone 020 8891 2007

EickLoxx Small Contour Plate Set




EICKLOXX SMALL CONTOUR PLATE SET – COMPONENTS AND CHARACTERISTICS

The EickLoxx Small Contour plate set includes various anatomically preformed osteosynthesis plates for the treatment of fractures of the distal radius and femur, ilium, acetabulum and for arthrodesis of the carpus and tarsus. It complements the cut-to-size plates of the EickLoxx Small osteosynthesis system and is therefore also designed for the treatment of cats and dogs up to 15 kg. The instruments from the EickLoxx Small osteosynthesis system are used in a similar way.

The titanium plates of the EickLoxx Small Contour plate set are available in two plate thicknesses: The 2.2 mm plates (gold, green and magenta) can be polyaxially locked with the 1.7 mm and 2.3 mm screws, and the 3.0 mm plates (light blue) can be polyaxially locked with the 2.7 mm, 3.5 mm and 4.0 mm screws from the EickLoxx series.

Despite contouring, the plates can be bent as required using the bending irons and / or the plate bending pliers with rollers and shortened using the cutting pliers.

The plate tray with the EickLoxx Small Contour plates fits into the container (Item No. 185555) on the EickLoxx Small Instrument and Implant Tray.

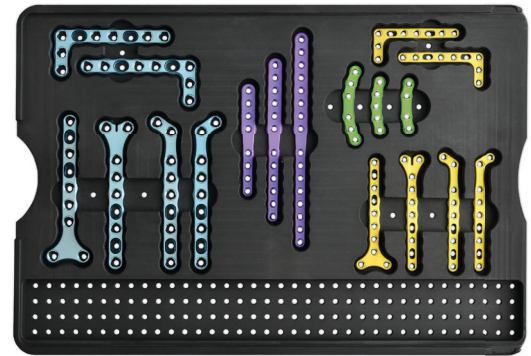

Titanium EickLoxx Small Contour Bone Plates

- ▶ Titanium alloy for maximum biocompatibility
- ▶ Multidirectional locking bi- and monocortical
- ▶ Eliminates abrasion
- ▶ Geometry optimised strength
- ▶ Bendable in two planes

197910 – 197915, 197920 – 197925, 197930 – 197935




Characteristics

- ▶ EickLoxx bone plates minimise contact with the periosteum and thus reduce the iatrogenic stress on bone perfusion that is common with conventional compression plates
- ▶ Preservation of perfusion significantly reduces the risk of infection and accelerates bone healing
- ▶ Resistance to infection is also increased by the biocompatibility of titanium and the absence of friction wear

Biological advantages

- ▶ Reduces damage to the vascularisation
- ▶ Increased resistance to infection
- ▶ Accelerated healing

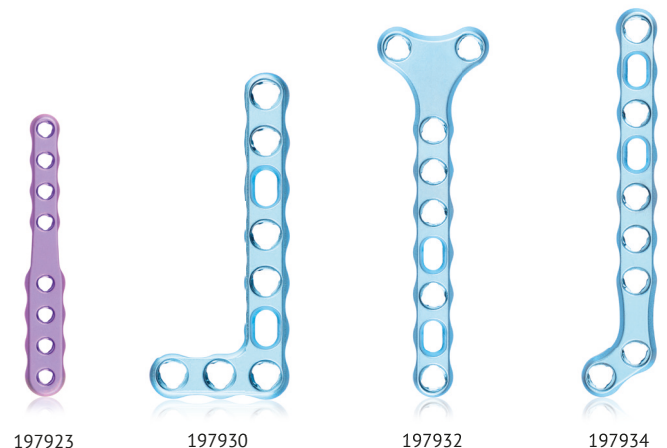

EICKLOXX SMALL CONTOUR PLATE SET – ITEM LIST

| EickLoxx Small Contour Plate Set | | |
|----------------------------------|--|----------|
| Item no. | Description | Quantity |
| 197900 | Complete set, consisting of: | |
| 197950 | EickLoxx Small Contour Plate Tray, without implants | 1 |
| 197910 | EickLoxx Small Contour Bone Plate, L-plate left, 9 holes, system 1.7 / 2.3 mm, gold, dimensions (in mm): L 39 x W 18 x H 2.2 | 1 |
| 197911 | EickLoxx Small Contour Bone Plate, L-plate right, 9 holes, system 1.7 / 2.3 mm, gold, dimensions (in mm): L 39 x W 18 x H 2.2 | 1 |
| 197912 | EickLoxx Small Contour Bone Plate, T-plate, 10 holes system 1.7 / 2.3 mm, gold, dimensions (in mm): L 54 x W 18 x H 2.2 | 1 |
| 197913 | EickLoxx Small Contour Bone Plate, Y-plate, 10 holes, system 1.7 / 2.3 mm, gold, dimensions (in mm): L 56 x W 18 x H 2.2 | 1 |
| 197914 | EickLoxx Small Contour Bone Plate, Hockey-plate left, 9 holes, system 1.7 / 2.3 mm, gold, dimensions (in mm): L 58 x W 11 x H 2.2 | 1 |
| 197915 | EickLoxx Small Contour Bone Plate, Hockey-plate right, 9 holes, system 1.7 / 2.3 mm, gold, dimensions (in mm): L 58 x W 11 x H 2.2 | 1 |
| 197920 | EickLoxx Small Contour Bone Plate, C-plate, 4 holes, system 1.7 / 2.3 mm, green, dimensions (in mm): L 28 x W 6 x H 2.2 | 1 |
| 197921 | EickLoxx Small Contour Bone Plate, C-plate, 5 holes, system 1.7 / 2.3 mm, green, dimensions (in mm): L 30 x W 6 x H 2.2 | 1 |
| 197922 | EickLoxx Small Contour Bone Plate, C-plate, 6 holes, system 1.7 / 2.3 mm, green, dimensions (in mm): L 40 x W 6 x H 2.2 | 1 |
| 197923 | EickLoxx Small Contour Bone Plate, Cricket-plate, 4+4 holes, system 1.7 / 2.3 mm, magenta, dimensions (in mm): L 55 x W 6–8 x H 2.2 | 1 |
| 197924 | EickLoxx Small Contour Bone Plate, Cricket-plate, 6+6 holes, system 1.7 / 2.3 mm, magenta, dimensions (in mm): L 80 x W 6–8 x H 2.2 | 1 |
| 197925 | EickLoxx Small Contour Bone Plate, Cricket-plate, 8+8 holes, system 1.7 / 2.3 mm, magenta, dimensions (in mm): L 105 x W 6–8 x H 2.2 | 1 |
| 197930 | EickLoxx Small Contour Bone Plate, L-plate left, 9 holes, system 2.7 / 3.5 mm, light blue, dimensions (in mm): L 56 x W 24 x H 3.0 | 1 |
| 197931 | EickLoxx Small Contour Bone Plate, L-plate right, 9 holes, system 2.7 / 3.5 mm, light blue, dimensions (in mm): L 56 x W 24 x H 3.0 | 1 |
| 197932 | EickLoxx Small Contour Bone Plate, T-plate, 9 holes, system 2.7 / 3.5 mm, light blue, dimensions (in mm): L 72 x W 22 x H 3.0 | 1 |
| 197933 | EickLoxx Small Contour Bone Plate, Y-plate, 9 holes, system 2.7 / 3.5 mm, light blue, dimensions (in mm): L 72 x W 18 x H 3.0 | 1 |
| 197934 | EickLoxx Small Contour Bone Plate, Hockey-plate left, 9 holes, system 2.7 / 3.5 mm, light blue, dimensions (in mm): L 75 x W 15 x H 3.0 | 1 |
| 197935 | EickLoxx Small Contour Bone Plate, Hockey-plate right, 9 holes, system 2.7 / 3.5 mm, light blue, dimensions (in mm): L 75 x W 15 x H 3.0 | 1 |

