

## Loupes / Lupenbrille 310600

## Loupe Light System / LED-Strahler für Lupenbrille 310601

**What could be a reason that the LED head lamp (Item No. 310601) is not working, although it was recently put into operation?**

The LED head lamp is supplied with demo batteries, which are not long-lasting. Replace them with high-quality alkaline batteries or rechargeable batteries.

**Woran kann es liegen, dass der LED-Strahler (Art. Nr. 310601) nicht funktioniert, obwohl er erst seit Kurzem in Betrieb ist?**


Die LED-Strahler wird mit Demobatterien geliefert, welche nicht sehr langlebig sind. Tauschen Sie diese durch hochwertige Alkaline-Batterien oder durch Akkus aus.

**The glasses fall off when tilting the head down.**

The temple tips of the loupes (Item No. 310600) must be adapted to your head. In order to do this bend the temple tips down.

**Die Brille fällt herunter, wenn man den Kopf nach unten beugt.**

Die Brillenbügel der Lupenbrille (Art. Nr. 310600) müssen auf Ihren Kopf passend eingestellt werden. Verbiegen Sie dazu die Bügel nach unten.

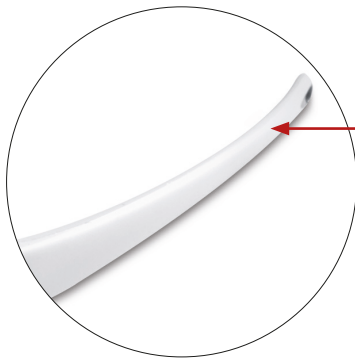

**FALSCH/WRONG**  
 Lieferzustand /  
 condition of delivery

**RICHTIG/CORRECT**  
 Nach Anpassung /  
 After adaption



**EICKEMEYER®**

**GERMANY** | EltstraÙe 8 | 78532 Tuttlingen | Deutschland | T +49 7461 96 580 0 | F +49 7461 96 580 90 | info@eickemeyer.de | www.eickemeyer.de

**SWITZERLAND** | Sandgrube 29 | 9050 Appenzell | Schweiz | T 071 788 23 13 | F 071 788 23 14 | info@eickemeyer.ch | www.eickemeyer.ch

**UNITED KINGDOM** | 3 Windmill Business Village | Brooklands Close | Sunbury-on-Thames | Surrey | TW16 7DY | UK | T 020 8891 2007 | www.eickemeyer.co.uk

**DENMARK** | Solbakken 26 | Hammelev | 6500 Vojens | Danmark | T +45 70 20 50 19 | info@eickemeyer.dk | www.eickemeyer.dk

**NETHERLANDS** | Bellweg 44 | 4104 BJ Culemborg | Nederland | T +31 345 58 9400 | info@eickemeyer.nl | www.eickemeyer.nl

**ITALY** | Via G. Verdi 8 | 65015 Montesilvano (PE) | Italia | T 085 935 4078 | F 085 935 9471 | info@eickemeyer.it | www.eickemeyer.it

**POLAND** | Al. Jana Pawła II 27 | 00-867 Warszawa | Polska | T 22 185 55 76 | F 22 185 59 40 | info@eickemeyer.pl | www.eickemeyer.pl

**CANADA** | 617 Douro Street, Suite #205 | Stratford | Ont. | Canada | N5A 0B5 | T +1 519 273 5558 | F +1 519 271 7114 | info@eickemeyervet.ca | www.eickemeyercanada.ca